

IDENT-ALERT® COLOR CODED SNAPS

Identifying special risk patients is a snap!

Accurate patient classification is a snap with Ident-Alert® Color Coded Snaps from Precision Dynamics. Use with PDC snap closure wristbands to quickly and cost-effectively identify special patient conditions.

Ident-Alert Color Coded Snaps:

- ➕ Provide convenience by allowing caregivers to classify multiple patient conditions—all on one PDC wristband!
- ➕ Soft, flexible material with rounded edges for patient comfort.
- ➕ Enhance patient safety with large, easy-to-read text that identifies at-risk patients.
- ➕ Locking SecurSnap® safety feature prevents transfer of alert to another patient.
- ➕ If there is a change in a patient's condition, the Ident-Alert® Snap can be removed using safety scissors without destroying the admitting wristband.

PDC offers a comprehensive line of color coded alert indicators which include: Alert Bands®, Ident-Alert® Color Coded Snaps, and Ident-Alert® Wristband Companion Labels. See our website and Wristband & Label Systems Catalog for full details.

"We were already using PDC wristbands for patient ID when one of our nurse clinicians discovered PDC's alert clips. We liked the idea because you could use multiple alerts on a single wristband. The lettering really stands out which allows all our nurses, even those with color blindness, to easily read the alerts. The soft and flexible material is also comfortable for our patients. Ident-Alert color coded snaps help us support the national color coded wristband standardization initiative."

—Yvonne Tanuz-Trujillo, BS, PT, CWS, FCCWS | UNMH Value Analysis Facilitator
UNM Hospitals, University of NM Health Sciences Center | Albuquerque, NM

TURN OVER TO VIEW PRODUCT ORDERING INFORMATION.



Identify special risk patient conditions



Large text is easy to read



Fits PDC snap closure wristbands

3 EASY STEPS TO APPLY IDENT-ALERT® COLOR CODED SNAPS!

STEP 1



Push snap and peel out of carrier.

STEP 2



Place stud side of snap through hole in wristband.

STEP 3



Fold snap up to securely close.



Can use multiple alerts on single band.



POSITIVE ID. POSITIVE OUTCOMES.™

VALENCIA, CA
25167 Anza Drive
Valencia, CA 91355
Tel 800.435.4242 or 661.257.0233
Fax 800.321.4409
Email info@pdcorp.com
www.stjohninc.com

SAN FERNANDO, CA
13880 Del Sur St.
San Fernando, CA 91340
Tel 800.772.1122 or 818.897.1111
Fax 818.686.9314
Email info@pdcorp.com
www.pdcorp.com

EUROPE, MIDDLE EAST & AFRICA
Rue de l'Industrie 17
1400 Nivelles, Belgium
Tel +32 67 89 56 56
Fax +32 67 21 77 10
Email info@pdc-europe.com
www.pdc-europe.com

HC-144-0911 250

ORDERING INFORMATION

IDENT-ALERT® 8620

EASY TO READ ALERTS
Shown Actual Size



PART #	8620A-16-PDO	8620F-14-PDO	8620D-33-PDO	8620LT-22-PDO	8620L-37-PDO
ALERT	ALLERGY	FALL RISK	DNR	NO LATEX	LIMB ALERT
COLOR	Red	Yellow	Purple	Green	Day Glow Pink
SNAPS/ PACK	240	240	240	240	240
USAGE	Best if used with following PDC bands: 120, 126, 140, 420, 440, 520, 529, 540, 860, 940, 5020, 5040, 5050, 5060, 5070, 5080, 5090, 5120, 5140, TM-SB-29295, TM-OP-29295				

IDENT-ALERT® 8621

EASY TO READ ALERTS
Shown Actual Size



PART #	8621A-16-PDH	8621F-14-PDH	8621D-33-PDH	8621LT-22-PDH	8621L-37-PDH
ALERT	ALLERGY	FALL RISK	DNR	NO LATEX	LIMB ALERT
COLOR	Red	Yellow	Purple	Green	Day Glow Pink
SNAPS/ PACK	200	200	200	200	200
USAGE	Best if used with following PDC bands: 7020, 7023, 7023S, 7024, 7024S, 7029S, 7041, 7041S, 7043, 7043S, 7049, 7049S, 7055, 7340, 7341, 7342, 7050, 7056, 7550, 9320, 9325, 9326, 9327, 9330, 9332, 9333				

HC-144-0911 250



VALENCIA, CA
25167 Anza Drive
Valencia, CA 91355
Tel 800.435.4242 or 661.257.0233
Fax 800.321.4409
Email info@pdcorp.com
www.stjohninc.com

SAN FERNANDO, CA
13880 Del Sur St.
San Fernando, CA 91340
Tel 800.772.1122 or 818.897.1111
Fax 818.686.9314
Email info@pdcorp.com
www.pdcorp.com

EUROPE, MIDDLE EAST & AFRICA
Rue de l'Industrie 17
1400 Nivelles, Belgium
Tel +32 67 89 56 56
Fax +32 67 21 77 10
Email info@pdc-europe.com
www.pdc-europe.com

POSITIVE ID. POSITIVE OUTCOMES.™