

SECURLINE® BAR CODE BLOOD BAND INSTRUCTIONS

1. Lift self-laminating, clear shield from the patient information area on the Bar Code Blood Band by holding the strap at the perforation.



2. Place your preprinted patient label (not included) in the center of the wristband information area.



3. Peel away white liner from the clear adhesive shield flap. Press clear cover flap down firmly from left to right, on a flat surface several times.



4. Separate the draw tube tail from the Bar Code Blood Band.



5. Apply Bar Code Blood Band to patient, allowing one finger space between wrist and wristband closure.



6. Apply preprinted patient label to draw tube. We recommend V-Notch Secondary Labels.



7. Peel liner away from tube tail and attach to first draw tube.



For more information about Securline® Bar Code Blood Band, Please contact Precision Dynamics at **800.772.1122** or 818.897.1111 pdcorp.com/healthcare

HC-113-0710

